

making a difference



**WE TAKE
CARE
OF YOUR
SAFETY**

VACUETTE®

SAFETY Blood
Collection Sets


Our innovations for your safety.



greiner
BIO-ONE

The SAFETY Blood Collection Set offers the patient particularly gentle collection comfort.

The ergonomic product is individually packed. It is easy to open and produces little waste.

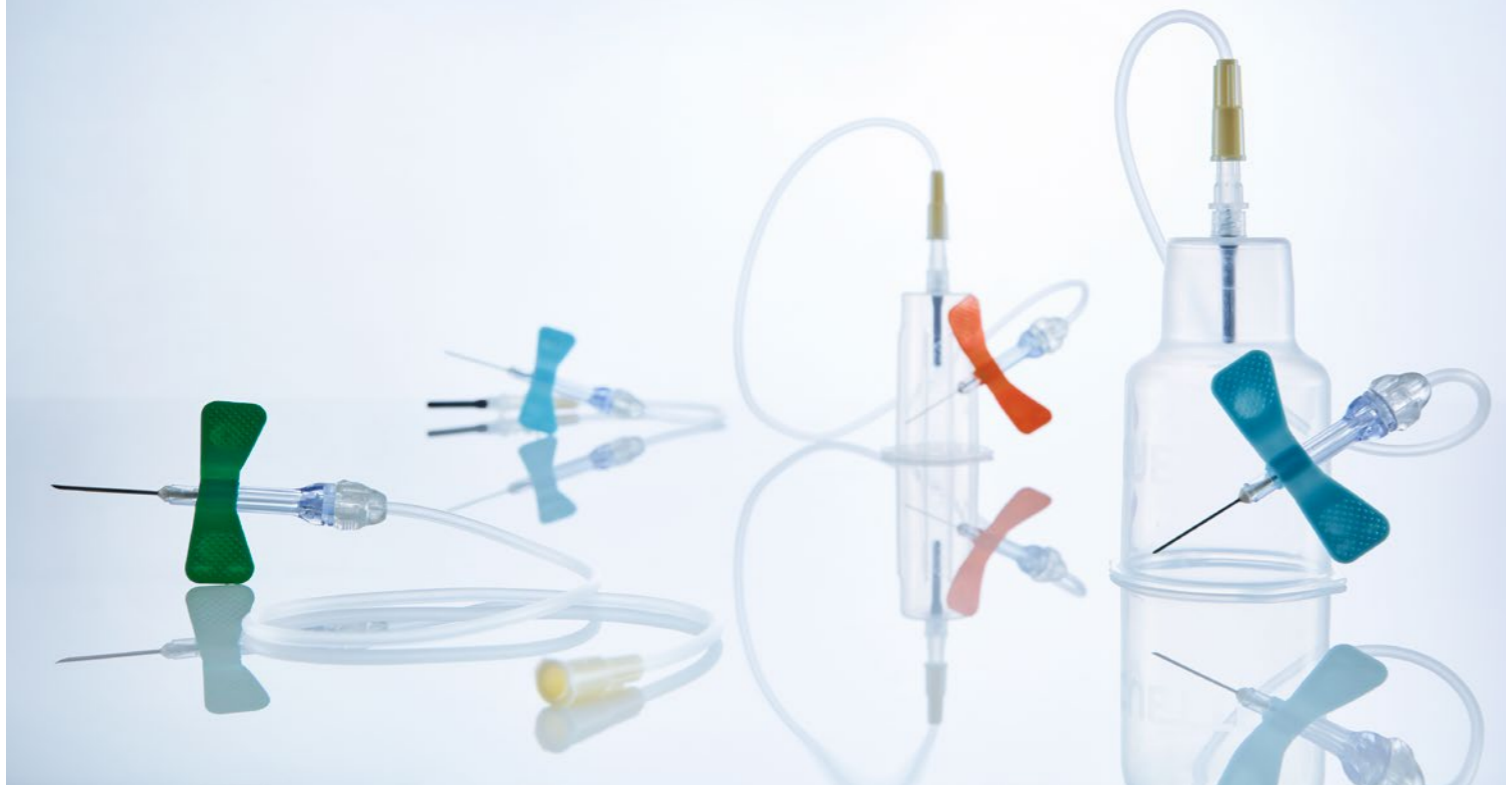

COVERING YOUR NEEDS FOR A RANGE OF VEIN CONDITIONS

VACUETTE®
SAFETY Blood Collection Sets

SPECIALLY FOR DIFFICULT VEIN CONDITIONS

The SAFETY Blood Collection Set has been specially developed for patients with difficult vein conditions, but is equally suitable for any regular blood taking situation, due to the gentle comfort it provides to a patient.


As the protection mechanism is activated whilst the needle is

still in the vein, a high level of safety is ensured. Correct activation via retraction is indicated by an acoustic signal.

Protect yourself and your employees from the risk of needlestick injury. Every needlestick injury is one too many.

INTUITIVE PRODUCT DESIGN.

- / The transparent view window provides clear vein entry indication.
- / Simple activation of the safety mechanism by lightly pressing in both sides of the hub. Correct activation is indicated by an acoustic signal.
- / It can be used for blood collection or infusion for up to a maximum of five hours.*
- / Assorted tubing length options offer more flexibility for blood collection.

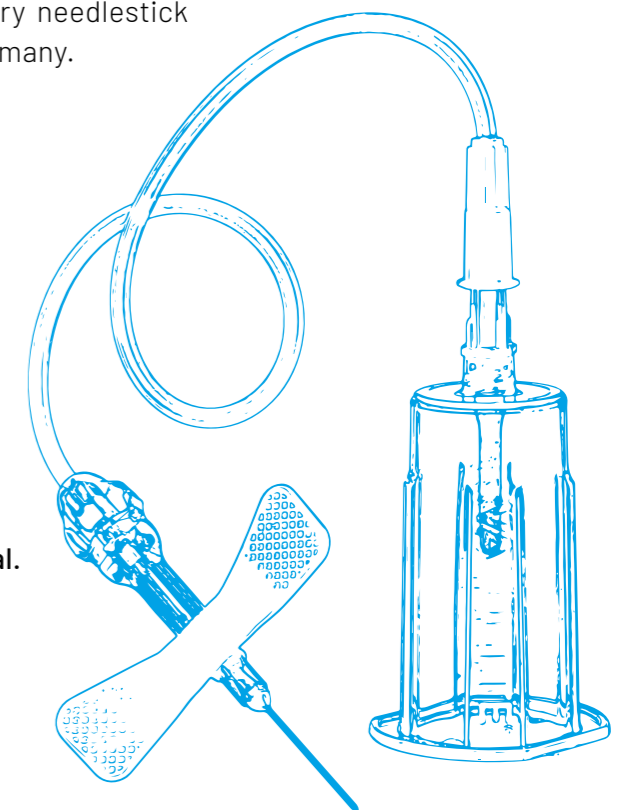

* Except SAFETY blood collection set with the blood culture holder

SAFETY BLOOD COLLECTION SETS COMBINATION PRODUCTS

As a practical combination product, the SAFETY Blood Collection Set is also available pre-assembled and packed in sterile blister pack.



SAFETY Blood Collection Set + Luer Adapter.

Pre-assembled with Luer adapter in sterile blister pack for hygiene and convenience.

Handling is optimised.



SAFETY Blood Collection Set + Holder.

No threading of needle necessary.

A practical, ready-to-use solution that saves time.

Packed in a sterile blister.



SAFETY Blood Collection Set + Blood Culture Holder.

Pre-assembled for blood culture collection.

Compatible with common-wide-necked culture bottles as well as evacuated blood collection tubes.



Only one venepuncture necessary for culture and blood collection.

Packed in sterile blister for convenience.

***PLEASE OBSERVE THE INSTRUCTIONS FOR USE AT**

www.gbo.com

INSTRUCTIONS FOR USE*

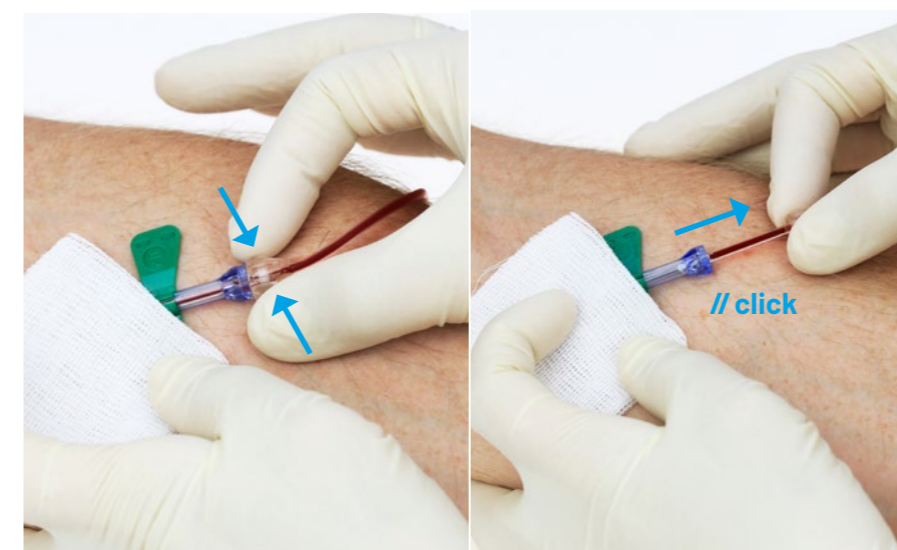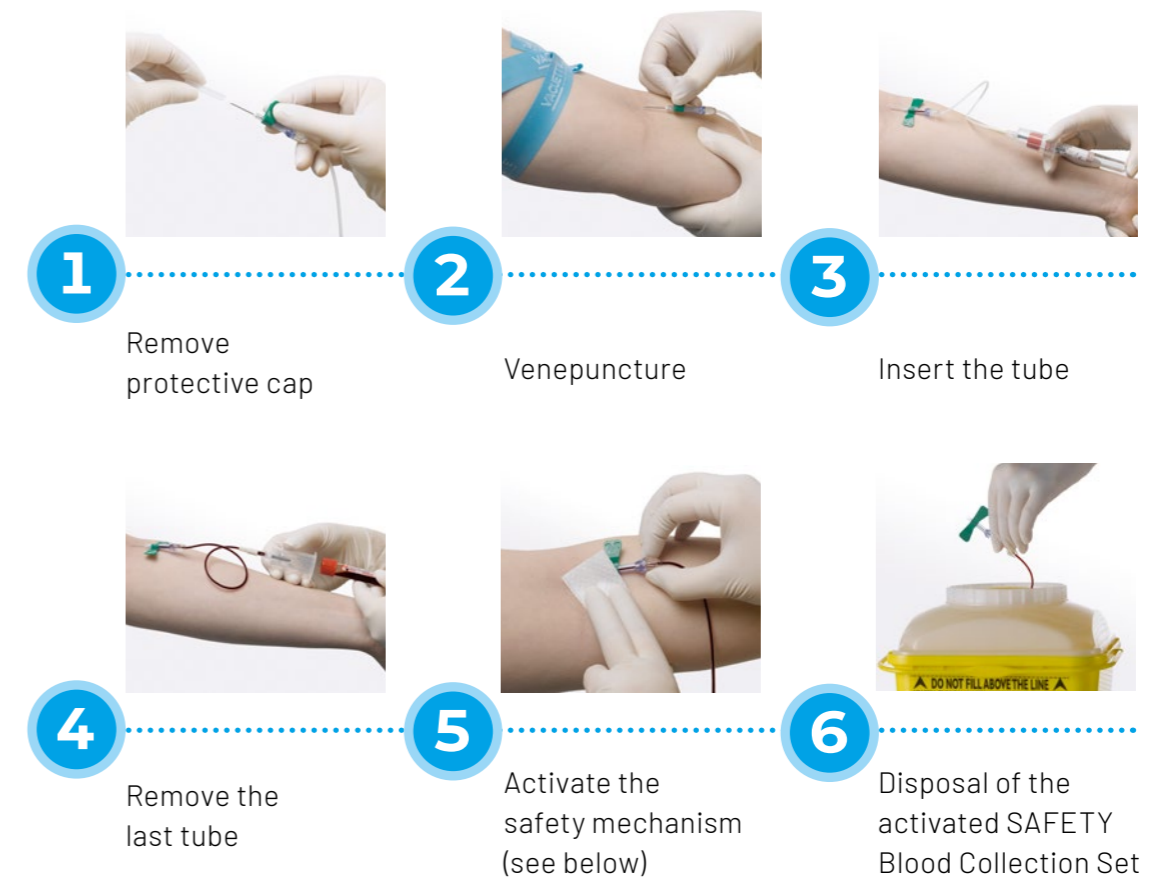

ACTIVATION OF THE SAFETY MECHANISM

By pressing lightly on both sides of the shield.

Then pull back the transparent hub. Correct activation is indicated by an audible click.



SAFETY FOR EVERY LEVEL

You will find the right product in our range for every level of safety. From entry-level products to premium safety products with a semi-automatic safety mechanism.



PERSONAL ADVICE

Our sales representatives will be happy to advise you at any time if you have any questions about our products and their application. Together you will find the right product for your individual needs.



TRAINING OPPORTUNITIES

We offer a wide range of training options for the correct handling of our products. In person or virtually with our application experts or in webinars or e-learning courses online on our website.

PRODUCT OVERVIEW

SAFETY Blood Collection/Infusion Set

Item no.	Colour code	Needle size	Needle length	Needle wall type	Tubing length	Tubing length	Qty. inner / outer
450091V1	● green	21 G	19 mm	thin wall	7.5"	19 cm	90 / 1,080
450191V1	● green	21 G	19 mm	thin wall	12"	30 cm	90 / 1,080
450092V1	● blue	23 G	19 mm	thin wall	7.5"	19 cm	90 / 1,080
450090V1	● orange	25 G	19 mm	thin wall	7.5"	19 cm	90 / 1,080

SAFETY Blood Collection Set + Luer Adapter

Item no.	Colour code	Needle size	Needle length	Needle wall type	Tubing length	Tubing length	Qty. inner / outer
450083V1	● green	21 G	19 mm	thin wall	4"	10 cm	90 / 1,080
450081V1	● green	21 G	19 mm	thin wall	7.5"	19 cm	90 / 1,080
450095V1	● green	21 G	19 mm	thin wall	12"	30 cm	90 / 1,080
450084V1	● blue	23 G	19 mm	thin wall	4"	10 cm	90 / 1,080
450082V1	● blue	23 G	19 mm	thin wall	7.5"	19 cm	90 / 1,080
450096V1	● blue	23 G	19 mm	thin wall	12"	30 cm	90 / 1,080
450099V1	● orange	25 G	19 mm	thin wall	12"	30 cm	90 / 1,080

SAFETY Blood Collection Set + Holder

Item no.	Colour code	Needle size	Needle length	Needle wall type	Tubing length	Tubing length	Qty. inner / outer
450087V1	● green	21 G	19 mm	thin wall	4"	10 cm	35 / 420
450085V1	● green	21 G	19 mm	thin wall	7.5"	19 cm	35 / 420
450160V1	● green	21 G	19 mm	thin wall	12"	30 cm	35 / 420
450088V1	● blue	23 G	19 mm	thin wall	4"	10 cm	35 / 420
450086V1	● blue	23 G	19 mm	thin wall	7.5"	19 cm	35 / 420
450161V1	● blue	23 G	19 mm	thin wall	12"	30 cm	35 / 420

SAFETY Blood Collection Set + Blood Culture Holder

Item no.	Colour code	Needle size	Needle length	Needle wall type	Tubing length	Tubing length	Qty. inner / outer
450182	● green	21 G	19 mm	thin wall	7.5"	19 cm	24 / 240
450183	● blue	23 G	19 mm	thin wall	7.5"	19 cm	24 / 240
450184	● green	21 G	19 mm	thin wall	12"	30 cm	24 / 240
450185	● blue	23 G	19 mm	thin wall	12"	30 cm	24 / 240

making a difference

www.gbo.com

GREINER BIO-ONE GMBH
KREMSMÜNSTER, AUSTRIA

PHONE +43 7583 6791-0
FAX +43 7583 6318
E-MAIL office@at.gbo.com



GREINER BIO-ONE IS A GLOBAL PLAYER.
FIND THE CONTACT DETAILS OF YOUR
LOCAL PARTNER ON OUR WEBSITE.



This product information is addressed exclusively to healthcare professionals. Devices of Greiner Bio-One are to be used by properly trained healthcare professionals only in accordance with the relevant Instructions for Use (IFU). For a listing of indications, contraindications, precautions and warnings, please refer to the Instructions for Use which accompanies each product or is available for download from our website at www.gbo.com (Download Center). For more information contact your local Greiner Bio-One sales representative or visit our website.

All information is provided without guarantee despite careful processing. Any liability, warranty or guarantee of Greiner Bio-One GmbH is excluded. All rights, errors and changes are reserved. If not stated otherwise, Greiner Bio-One GmbH has all copyrights and/or other (user-)rights in this documents, in particular to signs such as the mentioned (word-picture-)brands and logos. Any use, duplication or any other use of the rights of Greiner Bio-One GmbH is expressly prohibited.
Media owner: Greiner Bio-One GmbH / **Manufacturer:** [Samson Druck GmbH / 5581 St. Margarethen]

980157 en [rev.03.09.2024]


greiner
BIO-ONE