



IMPROVED VASCULAR NAVIGATION WITH CATHCOMPASS™

CHRISTIE+

Exclusive to

VEINVIEWER® Flex and Vision2

CathCompass™, is an integrated reference graphic projected onto the skin to enable a real-time comparison of the patient's veins to the standard colour coded peripheral IV catheter (cannula) gauge sizes. This provides the clinician with guidance to help choose the optimal PIVC for the patient's veins.

As it is common to need more than one attempt when having a PIVC inserted, using VeinViewer® to assess the veins is a considerable contribution to ensuring a comfortable procedure and successful outcome as well as reducing stress for the patient.

PIVCs come in various dimensions to meet clinical needs, and patients' vein conditions. A PIVC should occupy no more than 1/3 of total vein size to enable adequate blood flow around the catheter and reduce risk of phlebitis.




VEINVIEWER®.
FOR ACCURATE, REAL-TIME
VISUALIZATION OF VEINS UP TO
10MM BELOW THE SKIN.

- / Universal / fine detail / inverse picture modes to adapt to patient skin type and conditions
- / 3 picture sizes for focussing on smaller areas as required
- / MaxBright to ensure visibility even in the brightest conditions
- / Image capture to save images of blood patterns if required for documentation
- / Vision2: TriColor gives a choice of green, yellow or white image
- / CathCompass™ for catheter size references



FOR FURTHER INFORMATION AND/OR SAMPLE ORDERING
PLEASE VISIT OUR WEBSITE OR CONTACT US.

14G (orange)	=	2.2 mm
16G (grey)	=	1.7 mm
18G (green)	=	1.3 mm
20G (pink)	=	1.1 mm
22G (blue)	=	0.9 mm
24G (yellow)	=	0.7 mm

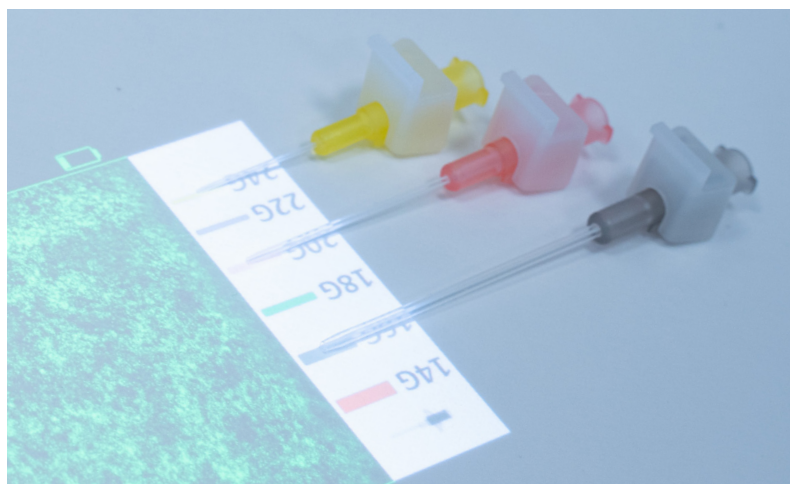

CathCompass™ is automatically activated, when the VeinViewer® is switched on. If not required, for example during a venous blood collection, it can be deactivated by pressing the function button.

If using the CathCompass™, use the VeinViewer® as normal, at a 90-degree angle to the surface of the patient's skin, at approximately 30cm distance, and use graphic as a reference guide to select the most suitable PIVC size.

PRODUCT OVERVIEW

VeinViewer® Flex

Item No.	Description	Inner / Outer [Qty.]
890001	Provided with 2 pc. lithium batteries, a charging station, 2 flexible mounting arms and 2 clamps	-/1

VeinViewer® Vision2

Item No.	Description	Inner / Outer [Qty.]
890004	Contains 1 pc. Lithium Ion battery	-/1

VeinViewer® Accessories

Item No.	Description	Inner / Outer [Qty.]
890011	Charging station for rechargeable battery for VeinViewer® Flex	-/1
890011A	Adapter for charging station for rechargeable battery for VeinViewer® Flex	-/1
890010	Spare rechargeable battery for VeinViewer® Flex and Vision 2	-/1