


VEINVIEWER® QUICK GUIDE



VEINVIEWER® Flex

CONTROL PANEL: FUNCTIONS AND MODES

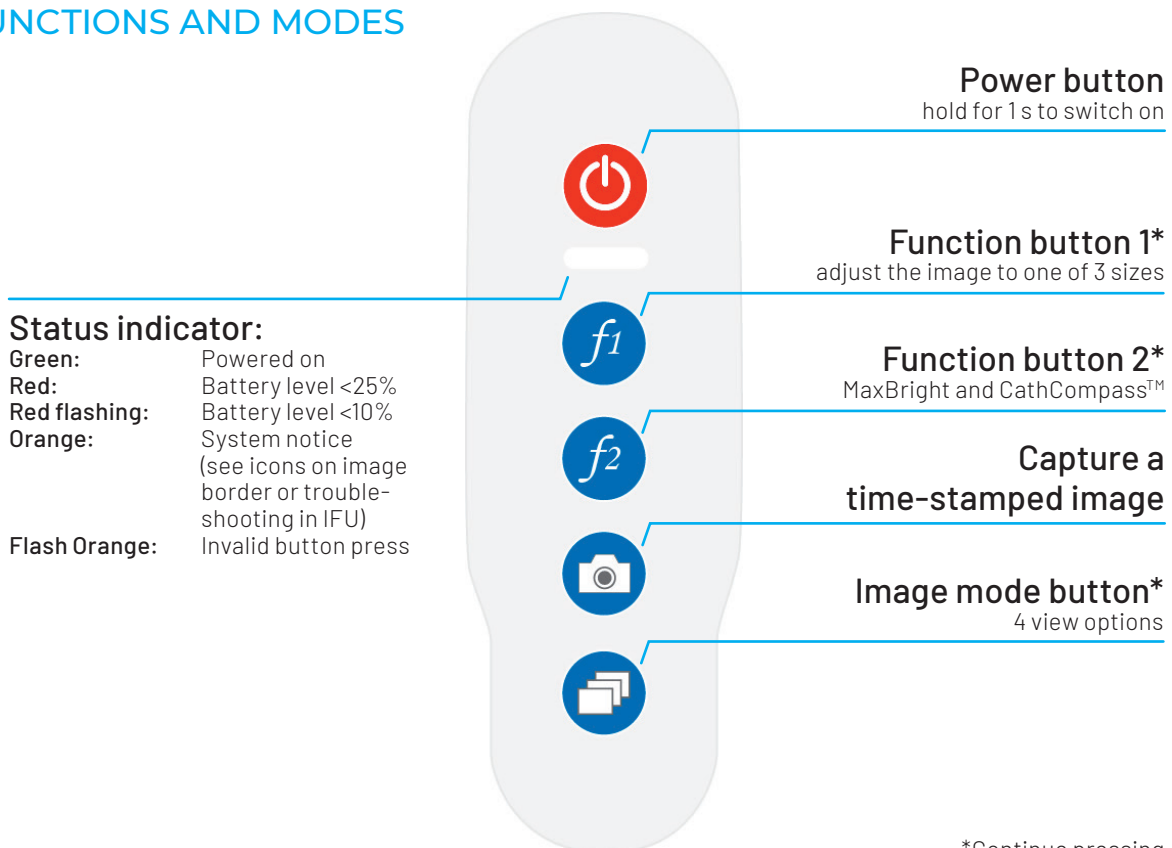

*Continue pressing to scroll through options.



VEINVIEWER®


Flex

VEINVISUALIZATION
IN A NEW LIGHT.

DETAILED FUNCTION DESCRIPTION



Resize (3 sizes available)

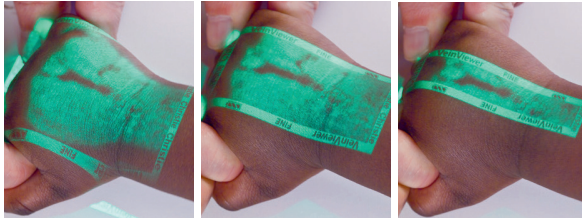


Photo

Capture a timestamped image of the vein conditions by pressing the camera function button. A camera symbol appears in the border to indicate a saved image, which can be transferred as PNG file to computer via the USB connection.




F2: MaxBright and CathCompass™

Max Bright (maximum brightness image)

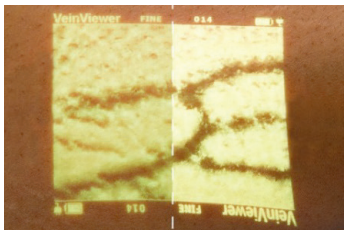

Helpful for use in environments with higher ambient light levels, such as sunlight.


CathCompass™

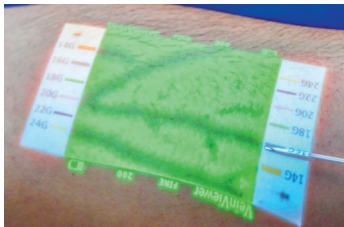


Enable or disable the catheter sizing reference graphic from the display.



Image Mode (4 options)

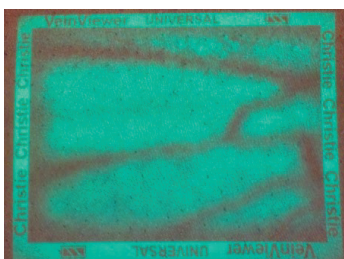

Universal mode appropriate for all patients

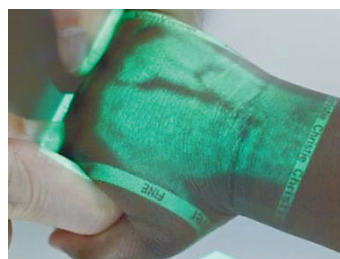

Fine detail mode for pediatrics, neonates, or sclerotherapy patients




Universal mode inverted



Fine detail mode inverted

CORRECT FOCUSING

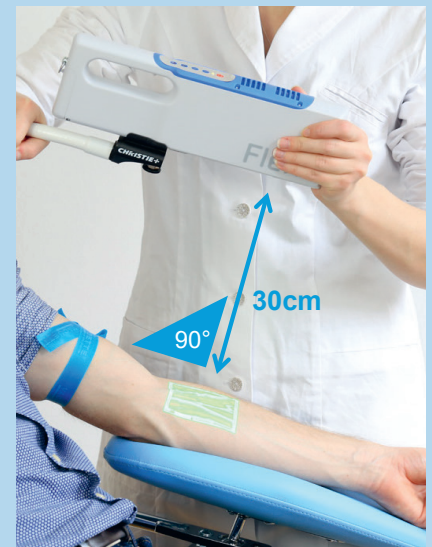

Correct focus position is indicated by the text in the image being clearly legible:

